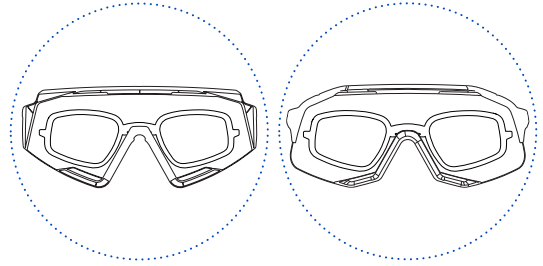
# OPTICAL CLIP

DESIGNED IN ITALY

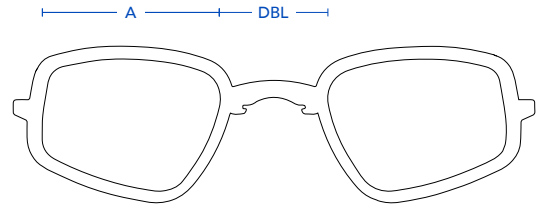
## OPTICAL CLIP FOR PRESCRIPTION LENS APPLICATION

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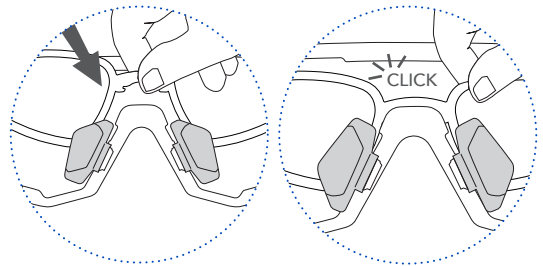
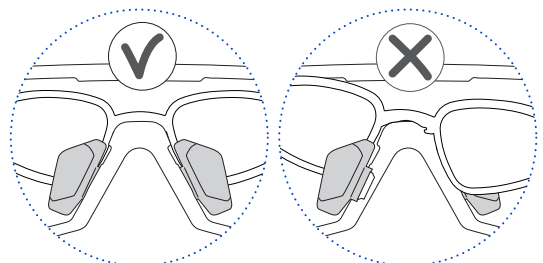
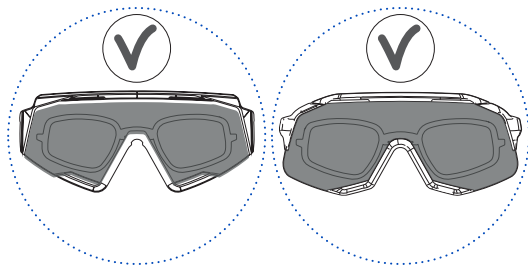
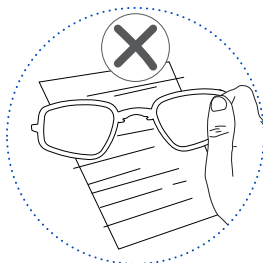
**EN Use:** class I medical device, as for Rule 1, Annex VII of EU Regulation 745/2017 on Medical Devices, whose intended use is the association with ophthalmic lenses for the correction of visual defects. Compliant with EU Regulation 745/2017. **Warning:** it cannot be excluded that sensitive or allergic subjects may experience adverse reactions in contact with the materials and treatments used. **Care and cleaning:** the clip-in must be stored at a temperature between -10 ° C and + 35° C. Avoid exposure to the sun in closed, unconditioned environments (e.g. car dashboard) because the temperatures reached in such conditions can damage its functionality. In case of contact with agents such as saltwater (seawater), clean immediately with fresh water and neutral soap. In case of wear, replacement is recommended. For cleaning, use a damp cloth and a normal neutral detergent, and dry with a clean soft cloth. Avoid the use of solvents (e.g. alcohol or acetone) and aggressive chemical detergents. **Marking:** model identification and date are printed on the label. **Complains:** if it is not possible to contact the store where it was purchased, for complaints or anomalies, send the request to [info@kooworld.cc](mailto:info@kooworld.cc)

// [kooworld.cc](http://kooworld.cc)

KASK S.P.A. Via Firenze 5, 24060 Chiuduno (BG) - ITALY

© 2021 KASK S.P.A. All rights reserved.

420272 - 02/2021

# KOOWORLD